



Patient
Information

Non-Surgical Periodontal Treatment

Say Goodbye
to Gum Disease

leading regeneration

Geistlich

Causes

Understanding gum disease: key facts

Why gum health matters

Periodontal disease, also known as gum disease, is a chronic bacterial infection that can destroy oral tissue and bone. It is very common but often goes unnoticed.

Stay ahead of the issue

Without treatment, gum disease can lead to tooth loss. However, timely therapy and regular check-ups can prevent serious interventions such as surgeries.

Risk factors

Smoking, poor oral health habits, poor nutrition, diabetes, and age can affect gum health. Taking proactive steps can make a big difference.



Healthy tooth and gum tissue

Healthy smile: Gums and teeth in perfect harmony.



Early signs of gum disease (Gingivitis)

First warning: Inflammation and bleeding indicate early gum disease.



Advanced gum disease (Periodontitis)

Critical stage: Severe gum damage and bone structure affected.

Symptoms


Have you been diagnosed with gum disease?

Do you suffer from...?

- Red or swollen gums
- Pain or bleeding when brushing or flossing
- Discomfort when chewing
- A persistent bad taste in your mouth

... then **Pocket-X® Gel** can help you restore your gum health.

Pocket-X® Gel

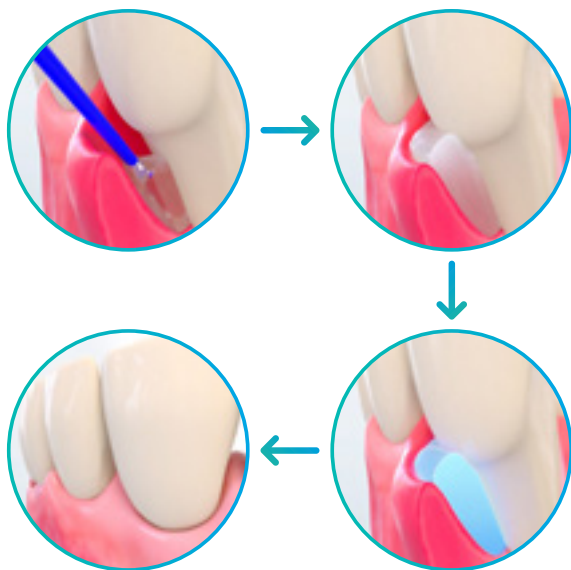

Results

- Gums enclose the tooth and prevent bacteria from entering
- Less inflammation
- Less bleeding

How can Pocket-X® Gel help?

Pocket-X® Gel was developed specifically to support the treatment of gum disease. It is applied after the periodontal gum pockets have been cleaned. The unique combination of ingredients in Pocket-X® Gel, such as hyaluronic acid, supports the healing of your gums and prevents new inflammation.¹

The benefits of Pocket-X® Gel

1. Gentle application

Easy and comfortable, even in hard-to-reach areas.

2. Convenient

The therapy does not require any additional time for you. It integrates seamlessly into your regular dental visit.

3. Effective protection

Forms a barrier to keep bacteria out, promoting healing.

4. Less inflammation and healthier gums

6 months after treatment with Pocket-X® Gel, there is a significant improvement in gum health.²

5. Comfortable experience

No unpleasant sensations, just effective care.

References

1 Prudentix Ltd., Israel, IFU 2023

2 Hirsch et al.: Clinical oral investigations vol. 26,4 (2022): 3721-3733

Pocket-X® Gel is a patented class IIa medical device (EU) made up of the following components:

Hyaluronic acid, Poloxamer 407, Octenidine and Phenoxyethanol. It is applied by your hygienist or dentist to support the healing phase after your visit.

Supporting your healing process

At the end of your treatment, you have had Pocket-X® Gel applied. Follow these simple aftercare instructions to maximize its benefits:



**Do not eat, rinse or gargle
for 2 hours after treatment**



**Do not drink any hot beverages
for 4 hours after treatment**



**Brush your teeth very gently
for 2 days after treatment**



**Do not use any interdental cleaning
accessories (i.e. floss or brushes)
for 2 days after treatment**



DOs

- Maintain good oral hygiene
- Follow your practitioner's recommendations
- Schedule follow-up appointments



Notes

- The information provided in this leaflet cannot replace the medical advice of your treating physician
- Follow the specific instructions of your doctor, dentist, or hygienist.

Geistlich

Manufacturer

Tree of Life Pharma Ltd.
POB 799
Lod 7110604 Israel

Made exclusively for

Prudentix Ltd.
POB 799
Lod 7110604 Israel

Distribution partner

Geistlich Pharma AG
Bahnhofstrasse 40
6110 Wolhusen
Switzerland
www.geistlich-pharma.com

For more details about our
distribution partners:
www.geistlich-dental.com

